

Instructions for Use Dalbo®-System

Dalbo® Abutment, Dalbo®-PLUS, Dalbo®-Classic, Dalbo®-B and Dalbo®-PLUS Gauge set

1 Scope of application of Instructions for Use

These Instructions for Use apply to the products listed under Point 29 in Table 1 and Table 2. The issuing of these Instructions for Use renders all previous versions invalid. The manufacturer rejects any liability for damages resulting from non-compliance with these Instructions for Use.

2 Trade name

See Point 29, Table 1 and Table 2

3 Intended use

The components are intended for use in prosthetic restorations on dental implants and root canal caps and to support procedures in the dental clinic or laboratory.

4 Expected clinical benefit

Restoration of chewing function and improved aesthetics.

The summary of safety and clinical performance (SSCP) for the implantable devices covered by these instructions for use, is available on our homepage, accessible at this address: <https://www.cmsa.ch/docs>.

5 Product description

Dalbo® System

The Dalbo® System is an anchorage for retentive, rigid or resilient use on implants and root canal caps. When restoring with one or two ball anchors, there is a certain degree of resilience; with three or more ball anchors, the denture is rigid.



Dalbo® Abutment

Implant ball anchorage with a ball head diameter of 2.25 mm. For available implant connections, see Point 29, Table 2.



Dalbo® CAD/CAM Retention element

Ball anchorage with threaded pins and a ball head diameter of 2.25 mm for screwing into milled bar constructions.



a)

b)

Dalbo® Male parts

Ball head diameter: 2.25 mm.

- a) The Elitor® (E) male part can be soldered or lasered onto root canal caps
- b) The Valor (V) male part can be cast-on during fabrication of the root canal cap




Dalbo® Female parts

All female parts of the Dalbo® System are suitable for ball male parts and implant ball head abutments with a diameter of 2.25 mm.



Elliptic variants

For an even stronger retention in the denture body. The elliptic retention wing can be reduced if necessary. However, any reduction leads to a loss of retention force.

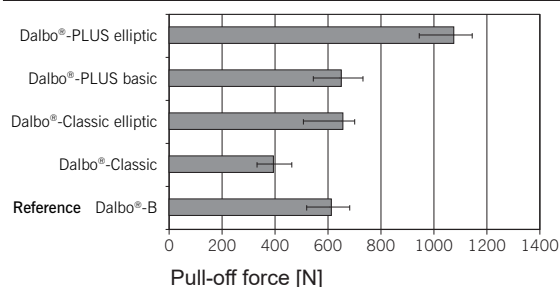

Comparison of retention force of Dalbo® Female parts in the denture body

All ball female parts demonstrate sufficient retention in the denture body under laboratory conditions. Remarkable is the high value of the Dalbo®-PLUS Female part elliptic, which in part even exceeds the mechanical properties of the resin.

Dalbo®-PLUS



Dalbo®-PLUS Female part TE basic

Is intended for assembly in the laboratory. It can be polymerised directly or bonded into a metal housing. The red duplicating aid enclosed with the product considerably simplifies the fabrication process of a bonding box in the laboratory: attach – block out undercut – duplicate – model – embed – cast – divest – blast – bond – finished!




Dalbo®-PLUS Female part TE elliptic

Is intended for placement directly in the patient's mouth. Experience has shown that the quality of the resin deteriorates with direct placement and that the female part can break from the denture under high loads. The elliptical design of the resin retention increases the retention force in the denture body.

Tuning system for female parts

The ball anchor is the oldest and most used anchoring principle. Minute differences in ball diameter, material selection, geometry and tolerance ranges influence the friction clearance. Two Dalbo®-PLUS Tuning female parts with different inner diameters of the lamellae retention inserts allow the retention force to be restored, regardless of the system used or even age-related wear.

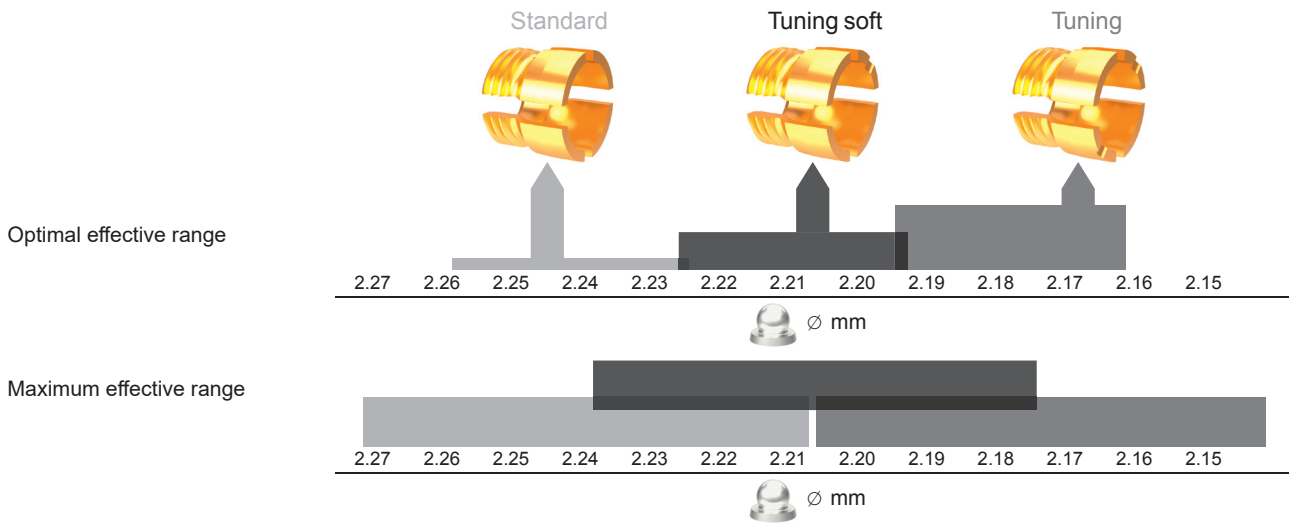

Lamellae retention inserts E

The lamellae retention insert is the actual retaining element of the system. It is made of Elitor® (E), a yellow precious metal alloy with ideal mechanical properties for a long-lasting and reliable function. Using a special screwdriver/activator (Cat. No. 072 609), the insert can be turned out of the housing and re-inserted again without removing the female part from the denture body. The two special tuning retention inserts with a reduced inner diameter provide for an exceptionally broad frictional clearance range and for restoration of the retention force even in case of already worn male parts.

Lamellae retention insert
(basal: no groove)



Standard

Tuning lamellae retention insert soft
(basal: 1 groove)



Tuning soft


Tuning lamellae retention insert
(basal: 2 grooves)



Tuning

The following abutment divergences can be compensated:

- root canal caps, depending on the activation of between 4° – 8°.
- on implants, depending on the activation, up to 20°.

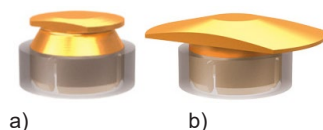**Dalbo®-Classic basic (a) / Dalbo®-Classic elliptic (b)**

The elliptic (b) version differs in the design of the female part from the basic (a) version by a massively pronounced, elliptically shaped resin retention, but without changing the popular, low insertion height. This design is indicated for direct placement in the mouth or quite simply where an extra strong retention force of the female part in the denture body is desired.

Version EV. The male part in the Valor (V) can be connected to the root canal cap by casting-on or soldering. Casting-on saves time and eliminates the need for jointing materials. The attached retention of both designs with an additional anti-rotation mechanism ensures a secure hold in the resin.

The EK version. The male part made of special burn-out Korak (K) resin results in a high-quality surface after casting when used correctly. The two female part designs are identical to the EV versions.

The following abutment divergences can be compensated:

- on root canal caps 10°
- on implants up to 20°

The Elitor® (E) female part is integrated exclusively by polymerisation.

**Dalbo®-B**

Was the world's first ball anchor and formed the basis for the development of the Dalbo®-PLUS and the Dalbo®-Classic.

The following abutment divergences can be compensated:

- on root canal caps 6°
- on implants up to 18°

The female part is integrated exclusively by polymerisation.

Auxiliary parts and instruments**Elastomer ring**

Dalbo®-Classic (Cat. No. 055 688)
Dalbo®-B (Cat. No. 051 005)

**Duplication aid G** (Cat. No. 072 626)

Can only be used for the Dalbo®-PLUS female part!
These «red» parts are slightly oversized with regard to the original components. This allows optimal bonding clearance for duplicating and bonding techniques. After being used as a duplication aid, the duplication auxiliary part can also be used as polishing protection.

**Spacer G**

Dalbo®-Classic (Cat. No. 072 625)
Dalbo®-B (Cat. No. 070 440)


Spacers basically replace the anchoring female parts during resin polymerisation in the laboratory. These are then removed from the fully polymerised denture. The most optimal polymerisation or bonding of the original female parts is performed by the dentist directly in the patient's mouth after cementation of the root canal caps. The spacer also proves to be an excellent polishing protection for the male part.

⚠ The duplicating aid and the spacer must not be used in place of the female part as a temporary replacement, nor for taking impressions in the mouth.


**Spacer disc Z** (Cat. No. 050394)

The tin spacer supplied with each female part enables vertical resilience. This is incorporated prior to polymerisation, after which it is removed again.

⚠ The tin spacer disc must not be inserted in the mouth.

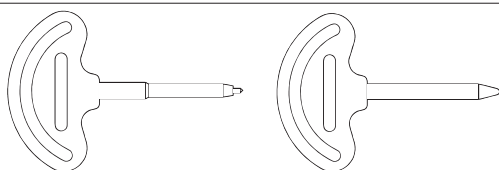
**Activator / deactivator**

The activator (Cat. No. 070197) and deactivator (Cat. No. 070199) for the Dalbo®-Classic and Dalbo®-B must not be sterilised. There is a risk that the resin handles will be destroyed during sterilisation in an autoclave. Please consider the published national guidelines when selecting a disinfection process and the Instructions for Use «Preparation of surgical and prosthetic products» (www.cmsa.ch/docs).

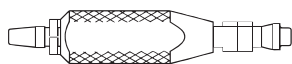
Punch for elastomer ring mounting (Cat. No. 070205)

For the optimal function and protection of the lamellae of the Dalbo®-Classic/elliptic and Dalbo®-B, the elastomer ring mounted on the female parts should not be removed.


Do not re-use elastomer rings once they have been slipped on.

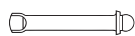


Transfer axis (Cat. No. 070157): for fabricating the master model.




Dalbo® Abutment Analog (Cat. No. 07000312): for master model fabrication for implant work.

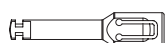


Dalbo® Abutment Screwdriver (Cat.No. 0700 0266):Screwdriver for Dalbo® Abutment.

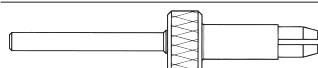

Special-Parallelometer Insert (Cat. No. 072 637) Is used for the parallel setting of several Dalbo® Male parts in the parallelometer.

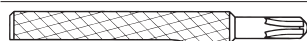

Screwdriver / Activator (Cat. No. 072 609): Is used to screw in and activate the lamellae retention insert for the Dalbo®-PLUS.

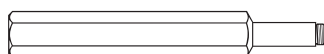

Heating Rod (Cat. No. 072 639); For extracting housings (only Dalbo®-PLUS) from the resin.

6 Indication

Removable, rigidly or resiliently anchored denture on implants and root canal caps:

- Hybrid prosthetics
- Unilateral dentures, splinted transversally
- Partial and free-end dentures in combination

7 Contraindications

- Unilateral, partial and free-end dentures without transversal support.
- Compensation of abutment divergences outside the respective application area of the system for male part and female part (see Point 5 Product Description)
- Restoration of severely periodontally damaged abutment teeth.
- Use on implant systems which are not approved for ball anchorages (see Point 29, Table 2)
- Use for the immediate restoration of implants if the manufacturer has not approved this indication.
- Lacking compliance of the patient with respect to correct follow-up / recall instructions.
- Patients with bruxism or other para-functional habits.
- In patients with allergies to one or more elements of the materials used in the product.
- Existing clinical picture in the patient's mouth does not permit the correct application of the products.

8 Compatible products

The Dalbo® Abutments have been designed for conventional implant interfaces. Approved systems are listed under Point 29, Table 2.

The Dalbo® Female parts are compatible with

- Dalbo®-Rotex
- Implant ball anchorage with a ball head diameter of 2.25 mm.
- Ball anchorage with a ball head diameter of 2.25 mm.

The retention force of the Dalbo® Female parts on the male parts of other manufacturers can vary due to their manufacturing tolerances and surface qualities.

9 User qualification

The expertise of a professional dentist or dental technician is required. The current Instructions for Use must be available at all times and be completely read and understood before the first application. The fabrication of dentures and their maintenance may only be performed by qualified specialists.

Only original tools and parts may be used for this work. For information and additional details, please contact your Cendres+Métaux SA representative.

Important information for the specialist

Warning symbol for increased caution

10 Prescription

Federal laws (USA) prohibit the use or sale by unlicensed dentists.

11 Side effects

- ⚠ This product may not be used in patients with allergies to one or more elements of the product materials. In patients with suspected allergy to one or more elements of the materials, this product may only be used following allergological clarification and proof of non-existence of an allergy.
Auxiliary instruments may contain nickel.
No known side effects if applied as intended.

12 Warnings

- ⚠ **Magnetic resonance environment**
The device has not been evaluated for safety and compatibility in the MR environment.
The product has not been tested for heating or migration in the MR environment.

13 General information

These Instructions for Use are sufficient for immediate application for the products described in this application area of the Instructions for Use. Dental or laboratory knowledge is required. Information: www.cmsa.ch/docs

14 Preventive measures

- ℹ – Only original tools and parts may be used for this work.
– The product components are supplied non-sterile. For more information see Section 16 Preparation.
– Secure parts against aspiration.
– Before any procedure, ensure that all required product components are available in sufficient quantity.
– For your safety, always wear suitable protective clothing.
– The mechanical cleaning of the product using a toothbrush and toothpaste may lead to premature wear of the functional parts.
– It is essential to block out undercuts prior to polymerising the female part.
– No pre-treatment required, such as sandblasting or silanisation of the housing of the female part.

15 Single use

Unless labelled otherwise, the product components are only intended for single use.
Products that are labelled for single-use are subject to a certain load during use, which can lead to wear, loss of function and/or malfunctions.

- ⚠ Re-use of products labelled as single-use products may compromise safety, function and performance.
Products for single-use have not been tested for re-use/preparation, which increases the risk of infection transmission.

16 Preparation

- ℹ After any fabrication or modification and prior to use, the prosthetic work, including all system components, must be cleaned, disinfected and, if appropriate, sterilised. Materials made of metal alloys, high-performance polymers (Pekkton®) and ceramics are suitable for steam sterilisation, whereas components made of plastic other than Pekkton® are not suitable. Consider published national guidelines when selecting a disinfection and sterilisation process and the Instructions for Use "Preparation of surgical and prosthetic products" (www.cmsa.ch/docs).


17 Scope of application

The Dalbo® System components are designed to fixate partial and full dentures on implants and root canal caps in the maxilla and mandible.


- ℹ We recommend that the denture be designed such that the largest possible support polygon can be achieved. Small distances between consecutive implants and long free-end saddles can cause undesirable effects such as increased wear of the system components.

18 Procedure**18.1 Fabrication of the primary reconstruction****18.1.1 Dalbo® Abutment**

- ℹ Before using the Dalbo® Abutment, follow the Instructions for Use of the implant manufacturer.

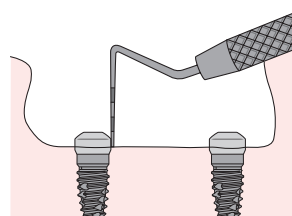

**Determining the abutment height**

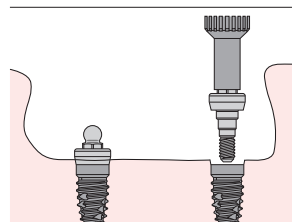Choose the required gingival height of the abutment height with a probe (1 mm graduation). The lower edge of the abutment should lie 1 mm above the gingiva. Various heights are available.

**Inserting the Dalbo® Abutment**

First, place the Dalbo® Abutment on the Dalbo® Abutment Screwdriver (Cat No. 07000266) and screw it into the implant by hand.

Then tighten with the torque ratchet with the corresponding torque (see information on the packaging). Make sure that the screwdriver is seated correctly on the abutment. Secure all parts against aspiration. After assembly, the screwdriver can be removed again by lifting it slightly.

-  – Screw in the abutment only once using the torque specified for this purpose.
- In case of immediate loading (observe indication of the implant manufacturer), ensure that the tightening torque of the abutment does not exceed that of the screwed-in implant. Recommendation, at least 5 Ncm less than the implant tightening torque.
-  The screwdriver features an ISO connection and fits onto the coupling inserts for the corresponding torque ratchets.

18.1.2 Dalbo® CAD/CAM Retention element

When modelling the bar in the CAD software, allow for the position of the Dalbo® CAD/CAM Retention element. A standard thread M2.0 is required for fixation of the Dalbo® CAD/CAM Retention element on the bar.

Inserting the Dalbo® CAD/CAM Retention element

After fabricating the milled bar, the Dalbo® CAD/CAM Retention element can be mounted on the bar using the Dalbo® Screwdriver (Cat. No. 0700 0266). Make sure that the screwdriver is seated correctly on the retention element. Secure all parts against aspiration. Then tighten with the torque ratchet with a torque of > 35 Ncm. After assembly, the screwdriver can be removed again by lifting it slightly.

-  The screwdriver features an ISO connection and fits onto the coupling inserts for the corresponding torque ratchets.

18.1.3 Dalbo® Male parts

Work preparation

Wax-up of the root canal cap with the root canal pin. In the case of several root canal caps, prepare the solder/laser surface at right angles to the direction of insertion. Use prefabricated, cast-on precious metal pins.

Inserting the Dalbo® Male part V by casting-on

Using the special parallelometer insert (Cat. No. 072 637) set the male part V as centrally as possible and wax it cleanly to the root canal cap. Then embed and cast.


Inserting the Dalbo® Male part V by soldering

Using the special parallelometer insert (Cat. No. 072 637) set the male part V as centrally as possible on the already cast and face-milled root canal cap and fix it with wax (take aesthetics into account). The solder gap should be continuous and between 0.05 – 0.20 mm wide. Design the soldering block, such that the male part V is securely held and good flame access is ensured.

Recommended solders: S.G 810 (Cat. No. 01000348) and S.G 750 (Cat. No. 01000345).

Inserting the Dalbo® Male part E by laser welding

In principle, only identical materials of the same composition should be joined together. Later failures can thus be reduced to an absolute minimum. We recommend using the Dalbo® Laser male part E (identical with Protor® 3) with the laser welding wire LW No. 5 (Cat. No. 0105 0041) and the casting alloy Protor® 3 (Cat. No. 010654) from Cendres+Métaux. Details on processing can be found in the Instructions for Use of the laser welding wires.


-  After thermal treatment (e.g. soldering, cast-on) slowly allow to cool down to room temperature to achieve the optimum mechanical properties without a tempering process. Fit a duplication aid or spacer to protect the male part during sandblasting and processing.


Inserting the Dalbo® Male part K by casting

Fill the cavity of the male part K with wax. Using the special parallelometer insert (Cat. No. 072 637) set the male part K as centrally as possible and wax it cleanly to the waxed-up root canal cap. After casting, polish the Dalbo® Male part extremely carefully and set to the desired frictional power with the female part.

18.2 Fabrication of the secondary reconstruction (denture)


General information


-  The tin spacer (Cat. No. 050 394) supplied with the female part enables vertical resilience. The soft spacer disc is placed over the entire root canal cap or anchoring element and adapted prior to polymerisation of the resin. After completion of the resin work, the spacer disc is removed again. Current clinical experience shows that the minimal vertical resilience disappears once the denture is placed. The greatest advantage is that the denture base is not overloaded on the root canal cap or the abutment.


-  For optimal function and protection of the lamellae, the elastomer ring mounted on the Dalbo®-Classic basic, Dalbo®-Classic elliptic and Dalbo®-B female parts should not be removed. If necessary, the elastomer ring can be changed using the punch (Cat. No. 070 205) as follows:


- 1) Remove blue sleeve from the punch
 - 2) Slip on several elastomer rings
 - 3) Mount sleeve
 - 4) By pushing the sleeve, the elastomer rings are pressed over the lamellae of the female part.
- Do not re-use elastomer rings once they have been slipped on.

-  Before fitting, protect the inside of the female part with Vaseline or soft silicone to prevent resin from penetrating.

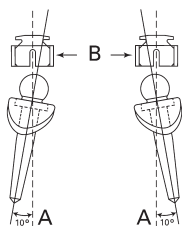

-  When fitting several female parts, make sure that they are positioned parallel to each other on the male parts and waxed firmly.

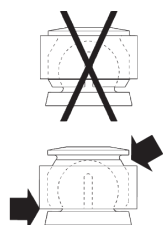

-  Fit the elastomer ring flush with the rim of the female part, so that the maximum retention can be used for the resin. Block out the undercuts and interpapillary spaces with impression plaster, wax, Flexistone or a rubber dam. Observe maximum abutment divergence. For the optimal function and protection of the lamellae the elastomer ring mounted on the Dalbo®-Classic and Dalbo®-B Female parts should not be removed.

Inserting the female parts in the laboratory by polymerisation


Block out the undercuts and interpapillary spaces with impression plaster, wax, Flexistone or a rubber dam.


Inserting the Dalbo®-PLUS basic female parts by bonding in the laboratory

The red duplicating aid G (Cat. No. 072 626) enclosed with the product is oversized in relation to the female part such that it creates an ideal bonding gap after casting the framework. After fabricating the primary construction, place the duplication aid G on the ball anchor, block out undercuts and duplicate the model (silicone type). After casting and finishing, clean the inner surface of the retention housing. Blast the outer surface of the Dalbo®-PLUS basic female part as well as the model cast housing with Al_2O_3 . Wax the female parts parallel to each other on the male parts and bond them into the framework. Use only suitable bonding agents.


Inserting the female part in the patient's mouth

To this end, the elliptic versions with reinforced resin retention are available specifically for Dalbo®-PLUS and Dalbo®-Classic. Create sufficient space prior to inclusion in the denture body. Fixate the elliptic female part in the mouth in parallel and block out the undercuts. If possible, drill an additional drainage canal through the denture body. With hybrid dentures, ensure that the root canal cap or the abutment are not loaded. This prevents the denture from rocking after insertion.

 Make sure that no resin has flowed into the housing of the female part. If necessary, remove the resin carefully and without damaging it so as not to impair the function of the female part.

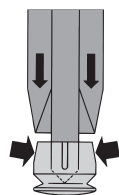18.3 Activation and deactivation**18.3.1 Dalbo®-Classic, Dalbo®-B****Activation:**

Press the four lamellae together evenly by applying light pressure with the activator provided for this purpose (Cat. No. 070 197).

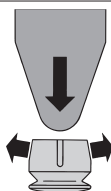**Deactivation:**

Apply light pressure with the deactivator provided for this purpose (Cat. No. 070199) to spread the lamellae evenly without the female part breaking out of the resin.

18.3.2 Dalbo®-PLUS

The Dalbo®-PLUS has three retention inserts, the lamellae retention insert E (Cat. No. 055643), the tuning lamellae retention insert soft E (Cat. No. 05000068) and the tuning lamellae retention insert E (Cat. No. 055687). For new work, the lamellae retention insert E (Cat. No. 055643) is used.

Our ball anchor diameter 2.25 mm has become the standard for most systems in the market. Experience and studies with third-party products have shown that minute differences, such as the choice of material, the geometry or the tolerance range, can reduce the frictional clearance of the Dalbo®-PLUS Lamellae retention insert. Two additional lamellae retention inserts are available to increase the clearance and for the after-care of already worn ball head male parts. Different notches on the lamellae make them easy to distinguish from «normal» retention inserts.

Standard Lamellae retention insert (without notches) normal friction


Tuning soft Lamellae retention insert (one notch) high friction

Tuning Lamellae retention insert (two notches) extra high friction




The screwdriver / activator (Cat.-No.072 609) is required to activate, deactivate and remove the lamellae retention insert. The instrument with four cams is pushed into the lamellae retention insert in its correct position up to the stop.


The retention force is adjusted by turning clockwise to increase the retention force and vice versa to reduce the retention force.

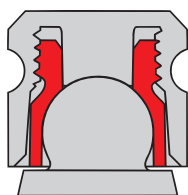

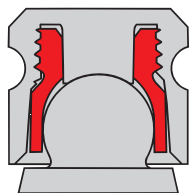

The basic setting on delivery is approximately 200 g, which also represents the minimum retention force to be set (lamellae retention insert is flush with the lower edge of the housing).

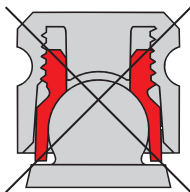


The retention force is adjusted by turning clockwise to increase the retention force and vice versa to reduce the retention force. The maximum retention force is approx. 1200 g (lamellae retention insert cannot be screwed in any further).



 The lamellae retention insert must not protrude from the housing, otherwise it could become detached from the housing by itself.

 After use, clean instruments with water and proceed as described in Section 16 (Preparation).

18.4 Follow-up

 Retaining elements in prosthetic work are subject to considerable stress in the mouth in a constantly changing environment, and thus more or less subjected to signs of wear. Wear is omnipresent in daily routine and cannot be avoided, only reduced. The amount of wear depends on the overall system. Our endeavours are aimed at using materials that are as optimally matched as possible in order to reduce wear to an absolute minimum. Proper seating of the dentures on the mucosa must be checked at least once each year, and relining carried out if required to prevent rocking movement (overload). We recommend checking the denture at intervals of approx. 3 months initially and to replace the retention inserts if necessary.

18.4.1 Modifications, relining


Before taking the impression, preferably remove the original female part from the denture.

Dalbo®-Classic / Dalbo®-B

Carefully remove the old female part from the denture.

Dalbo®-PLUS

1. Removing the lamellae retention insert
2. Screw the heating rod (Cat. No. 072 639) into the housing of the female part
3. Heat the opposite end over the flame of a Bunsen burner until the resin around the female part becomes soft.
4. Using pliers, pull the heating rod together with the female part out of the denture.
5. Before taking the impression, place the existing or a new Dalbo® Female part on the male part, the abutment. Place the master model analogue/transfer axis in the Dalbo® Female part to fabricate the master model.

 In the case of a bonded Dalbo®-PLUS Female part housing, the temperature required to neutralise the adhesive's bonding strength is many times higher!

18.4.2 Impression taking

Always use the original Dalbo® Female part for this purpose.

Place the Dalbo® Female part on the Dalbo® Male part or the Dalbo® Abutment.

Block out the space between the female part and the male part/abutment with a little soft wax before taking the impression. Ensure an exact fit, parallelism of the insertion direction and correct alignment with the occlusal plane of the female part. Take a functional impression. Use a solid impression silicone. Check that the material is distributed completely around the female part and that no impression material has flowed into the female part, otherwise clean the male part and female part and repeat impression taking.

18.4.3 Model fabrication

To fabricate the model, the implant analogue (Cat. No. 0700 0312) is used for an implant restoration, and the transfer axis (Cat. No. 070157) is used for root canal pin restorations by inserting it into the female part and fixing it securely. Then proceed with fabrication of the master model.

18.4.4 Insufficient denture retention – what to do:

1. Remove the denture, clean it and check which anchor system was used.
2. Check the denture to see if the female part or its components are damaged, replace if necessary and readjust denture retention.
3. Check whether the incorporated female part is correctly positioned on the male part. If this is not the case, the retention force is reduced and wear is very high. It is essential to insert the female part afresh.
The correct seating of the female part can be checked by using an easy-flowing silicone.
4. Check in the mouth to see if there are any signs of wear on the ball of the male part, which could be the cause of insufficient retention.
If yes, use the Dalbo®-PLUS Gauge set to check the level of wear in the patient's mouth. If the Dalbo®-PLUS was used, then denture retention can be easily readjusted by replacing the lamellae retention insert, which is available in 3 different force levels, in the housing.
If a different ball anchor system was used, we recommend changing to the Dalbo®-PLUS when the ball male parts are worn out.

18.4.5 Dalbo®-PLUS Gauge set

With the Dalbo®-PLUS Gauge set it is possible to check in the mouth whether the ball male part shows signs of wear.




Gauge set (Cat. No. 0700 0026).

Scope of delivery: male part gauge (Cat.-No. 0700 0027), female part gauge (Cat.-No. 0700 0024), lamellae retention insert (Cat.-No. 055 643), tuning lamellae retention insert soft (Cat.-No. 0500 0068), tuning lamellae retention insert (Cat.-No. 055 687), screwdriver / activator (Cat.-No. 072 609).



Male part gauge (Cat. No. 0700 0027).

Allows checking the set retention force in the denture outside the mouth. The nominal dimension of the ball diameter of the male part gauge is 2,245 mm and corresponds to the most commonly used systems on the market.

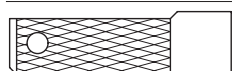

Female part gauge (Cat. No. 0700 0024)

Supplied without mounted lamellae retention insert.

Serves to check and determine the desired Dalbo®-PLUS Female part retention force in the patient's mouth and selection of the ideal lamellae retention insert.

Adjusting the retention force with the gauge set

1. Screw in the lamellae retention insert (Cat no. 055 643) with the screwdriver/activator (Cat no. 072 609) into the female part gauge (Cat no. 0700 0024) up to the 0-position (flush with the lower edge of the housing).

⚠ Repeated screwing and unscrewing of the lamellae retention insert reduces the protection against becoming loose by itself!

2. Check and adjust the retention force in the mouth with the female part gauge, which is secured with a thread, by increasing the retention force stepwise with $\frac{1}{4}$ turns. Adjustment is a matter of feeling. The recommended retention force lies between 600 – 900 g, but may vary depending on the number of anchors used and the patient situation.
3. If sufficient retention cannot be achieved with the lamellae retention insert, the tuning lamellae retention insert soft (Cat. No. 0500 0068) and later, the tuning lamellae retention insert (Cat. No. 055 687) are screwed into the female part gauge and the procedure is repeated.
4. Once the desired retention force has been reached, note the number of revolutions.
Unscrew the lamellae retention insert and adjust it again with the same number of turns in the original housing.
5. Note the type of lamellae retention insert used and the batch number in the patient file.

19 Materials

S = Syntax; TiAl6V4 ELI (Grade5)

T = Pure titanium (Grade 4); Ti > 98.9375 %

E = Elitor®; Au 68.60 %, Pt 2.45 %, Pd 3.95 %, Ag 11.85 %, Cu 10.60 %, Ir 0.05 %, Zn 2.50 %. $T_s - T_L$ 880 – 940 °C

V = Valor; Pt 89.0 %, Au 10.0 %, Ir 1.0 %. $T_s - T_L$ 1660 – 1710 °C

K = Korak; Residue-free burn-out resin for the casting technique.

G = Galak; Mouth-resistant resin

X = steel.

Detailed information on the materials and their classification is given in the specific material data sheets, the catalogue as well as the product list given in Table 1 in Point 29. See website www.cmsa.ch/docs or the Cendres+Métaux SA Dental Documentation (available free of charge from all Cendres+Métaux SA subsidiaries, branches and dealers).

20 Notes on storage

📖 The product must be stored in a dry place in its original packaging, at room temperature and without direct sunlight, unless otherwise stated on the packaging. Improper storage can influence the product properties and lead to failure of the restoration.

21 Patient information

21.1 Handling / follow-up

On the day of insertion of the dentures at the latest, the patient must be informed that regular follow-up care is necessary to maintain the health of the entire masticatory system and the functionality of the denture. Ensure that patients are motivated and instructed according to their own abilities such as manual dexterity and vision with regard to the handling and care of their teeth and dentures.

Permanent and removable dentures are subject to considerable stress in the mouth in a constantly changing environment, and thus more or less subjected to signs of wear. Wear is omnipresent in daily routine and cannot be avoided, only reduced. The amount of wear depends on the overall system.

Our endeavours are aimed at using materials that are as optimally matched as possible in order to reduce wear to an absolute minimum. Proper seating of the dentures on the mucosa must be checked at least once each year, and relining must be performed if required to prevent rocking movement (overload). We recommend checking the dentures at intervals of approx. 3 months initially and to replace the auxiliary parts such as retention inserts if necessary.

21.2 Insertion and removal of the dentures

Ensure that the dentures do not tilt, as any tilting can lead to damage. Never insert dentures by biting the teeth together. This can lead to damage or even breakage of the connecting element. Further information on handling and aftercare of dentures is available in the patient information brochure at www.cmsa.ch/docs.

Insertion

Hold the dentures between the thumb and forefinger, and place them back into the mouth on the anchors. Search or feel for the correct insertion position and push the dentures onto the anchors with gentle, steady pressure. Carefully close your jaws and check whether the dentures are in the correct final position.

Removal

Hold the dentures between the thumb and forefinger, and slowly, carefully and steadily pull them off the anchors and remove them from the mouth.

21.3 Cleaning and care

We recommend cleaning your teeth and your dentures after every meal. Cleaning of dentures includes cleaning of the connecting element. The gentlest cleaning is achieved by cleaning the connecting element under running water with a soft toothbrush. The most intensive cleaning is achieved when cleaning the dentures in a small ultrasonic device and adding a suitable cleaning agent. Never clean the high precision connecting elements with toothpaste. This could lead to damage. Caution should also be exercised in the case of unsuitable cleaning agents or tablets. This could also damage the high quality connecting element or impair its function. Only clean the connecting parts on the other teeth or implants with water and a soft toothbrush as well as an interdental brush. Do not use toothpaste to avoid damage.

For information and additional details, please contact your Cendres+Métaux SA representative.

22 Ordering information

More detailed information on the catalogue numbers, the number of products and their classification can be found in the product list under Point 29 in Table 1, the specific product catalogue, the packaging and, in the case of individual products, also directly on the product itself. You can find further information on the website www.cmsa.ch/docs or the Cendres+Métaux SA Dental Documentation (available free of charge from all Cendres+Métaux SA subsidiaries, branches and dealers).

For information and additional details, please contact your Cendres+Métaux SA representative.

23 Availability

Some of the products described in this document may possibly not be available in all countries.

24 Traceability batch number

The batch numbers of all parts used must be documented to ensure traceability. If different batch numbers are used for the products described in this application area of the Instructions for Use for the fabrication of dentures, all the batch numbers concerned must be recorded to ensure traceability.

25 Complaint

Cendres+Métaux SA must be notified immediately of any incident that has occurred with regard to the product to all branches, offices and dealers of Cendres+Métaux SA and, in case of serious cases, to the competent authority where the user is registered.

26 Safe disposal

The product must be disposed of in accordance with local laws and environmental regulations, taking into account the level of contamination. Cendres+Métaux LUX SA would be very pleased to accept precious metal waste. For information and additional details, please contact your Cendres+Métaux SA representative.

27 Trademarks

Registered trademarks of Cendres+Métaux Holding SA, Biel/Bienne, Switzerland include:

Dalbo® / Elitor®

Unless explained specifically, all products marked with "®" are not registered trademarks of Cendres+Métaux Holding SA, but registered trademarks of the respective manufacturer.

28 Disclaimer

The manufacturer rejects any liability for damages resulting from non-compliance with these Instructions for Use. This product is part of an overall concept and may only be used or combined with the corresponding original components and instruments. Otherwise, the manufacturer rejects any responsibility and liability. In case of complaints, please always include the batch number.

The use of third party products not distributed by Cendres+Métaux SA in connection with the products listed in Table 1 will void any warranty or other express or implied obligations of Cendres+Métaux SA.

The user of Cendres+Métaux SA products is responsible for determining whether or not a product is suitable for a specific patient and a specific situation.

Cendres+Métaux SA disclaims any express or implied liability and shall not be responsible for any direct, indirect, punitive or other damages arising from or in connection with errors in professional judgement or practice in the use or installation of Cendres+Métaux SA products.

The user is also obliged to regularly study the latest developments of the Cendres+Métaux SA products listed in Table 1 and their applications.

Please note: the descriptions contained in this document are not sufficient for the immediate application of Cendres+Métaux SA products. Specialist knowledge of dentistry, dental technology and instructions in handling the products listed in Table 1 by an operator with appropriate experience is always required.

29 Product list
Table 1

Cat. No.	Product name	Material	Single use	Labelling	Basic UDI-DI
See Table 2	Dalbo® Abutment	TiAl6 V4 ELI, (Grade5)	Yes	CE 0483	764016651000046E7
05002599	Dalbo® CAD/CAM Retention element	TiAl6 V4 ELI, (Grade5)	Yes	CE 0483	764016651000050DW
07000312	Dalbo® Abutment analogue	TiAl6 V4 ELI, (Grade5)	Yes	CE	764016651000034DY
07000266	Dalbo® Abutment screwdriver	Steel	No	CE	764016651000022DR
07000269	Torque ratchet including adapter	Steel	No	CE*	n/a (Third-party product)
07000268	Adapter for torque ratchet	Steel	No	CE*	n/a (Third-party product)
07000336	Adapter for special tools	Steel	No	CE*	n/a (Third-party product)
055750	Dalbo®-PLUS TEV basic	Ti / Elitor® / Valor	Yes	CE 0483	764016651000050DW
055889	Dalbo®-PLUS TEV elliptic	Ti / Elitor® / Valor	Yes	CE 0483	764016651000050DW
055752	Dalbo®-PLUS Female part TE basic	Ti / Elitor®	Yes	CE 0483	764016651000053E4
055890	Dalbo®-PLUS Female part TE elliptic	Ti / Elitor®	Yes	CE 0483	764016651000053E4
055643	Dalbo®-PLUS Lamellae retention insert E	Elitor®	Yes	CE 0483	764016651000053E4
05000214	Dalbo®-PLUS Tuning female part soft TE basic	Ti / Elitor®	Yes	CE 0483	764016651000053E4
05000215	Dalbo®-PLUS Tuning female part soft TE elliptic	Ti / Elitor®	Yes	CE 0483	764016651000053E4
05000068	Dalbo®-PLUS Tuning lamellae retention insert soft E	Elitor®	Yes	CE 0483	764016651000053E4
055771	Dalbo®-PLUS Tuning female part TE basic	Ti / Elitor®	Yes	CE 0483	764016651000053E4
055891	Dalbo®-PLUS Tuning female part TE elliptic	Ti / Elitor®	Yes	CE 0483	764016651000053E4
055687	Dalbo®-PLUS Tuning lamellae retention insert E	Elitor®	Yes	CE 0483	764016651000053E4
050394	Dalbo®-PLUS /-Classic /-B /-Z Spacer disc Z	Tin	Yes	CE	764016651000028E5
055647	Dalbo®-PLUS /-Classic Male part V	Valor	Yes	CE 0483	764016651000050DW
055921	Dalbo®-PLUS /-Classic /-B Lasering male part E	Elitor®	Yes	CE 0483	764016651000050DW
055760	Dalbo®-PLUS Housing of the female part T basic	Ti	Yes	CE 0483	764016651000053E4
055886	Dalbo®-PLUS Housing of the female part T elliptic	Ti	Yes	CE 0483	764016651000053E4
072626	Dalbo®-PLUS Duplication aid / spacer G	Galak	Yes	CE	764016651000006DT
070157	Dalbo®-PLUS /-Classic /-B /-Z / Profix Transfer axis	Steel	Yes	CE	764016651000032DU
072637	Dalbo®-PLUS /Dalbo®-Classic Special parallelometer insert	Steel	No	CE	764016651000018E2
072609	Dalbo®-PLUS Screwdriver / activator	Steel	No	CE	764016651000002DK
072639	Dalbo®-PLUS Heating rod	Steel	No	CE	764016651000010DJ
07000026	Dalbo®-PLUS Gauge set	Steel / Elitor®	No	CE	764016651000012DN
07000027	Dalbo®-PLUS Male part gauge	Steel	No	CE	764016651000011DL
07000024	Dalbo®-PLUS Female part gauge	Steel	No	CE	764016651000011DL
070222	KE forceps	Steel	No	CE	764016651000035E2
055689	Dalbo®-Classic EV basic	Elitor® / Valor	Yes	CE 0483	764016651000050DW
055701	Dalbo®-Classic EK basic	Elitor® / Korak	Yes	CE 0483	764016651000050DW
055892	Dalbo®-Classic EV elliptic	Elitor® / Valor	Yes	CE 0483	764016651000050DW
055893	Dalbo®-Classic EK elliptic	Elitor® / Korak	Yes	CE 0483	764016651000050DW
055698	Dalbo®-Classic Female part E basic	Elitor®	Yes	CE 0483	764016651000053E4
055887	Dalbo®-Classic Female part E elliptic	Elitor®	Yes	CE 0483	764016651000053E4
055688	Dalbo®-Classic Elastomer ring	Elastomer	Yes	CE 0483	764016651000053E4
055330	Dalbo®-Classic / Dalbo®-B Male part K	Korak	Yes	n/a	764016651000050DW
072625	Dalbo®-Classic Spacer G	Galak	Yes	CE	764016651000026DZ
070205	Dalbo® / Baer cylinder anchor punch, for elastomer ring mounting	Steel	No	CE	764016651000016DW
070197	Dalbo®-Classic /-B /-Z / Profix activator	Steel	No	CE	764016651000003DM
070199	Dalbo®-Classic /-B /-Z / Profix deactivator	Steel	No	CE	764016651000003DM
050427	Dalbo®-B EE	Elitor®	Yes	CE 0483	764016651000050DW
055331	Dalbo®-B EK	Elitor® / Korak	Yes	CE 0483	764016651000053E4
051511	Dalbo®-B Female part E	Elitor®	Yes	CE 0483	764016651000053E4
051005	Dalbo®-B Elastomer ring	Elastomer ring	Yes	CE 0483	764016651000053E4
050423	Dalbo®-B Male part E	Elitor®	Yes	CE 0483	764016651000050DW
070440	Dalbo®-B Spacer G	Galak	Yes	CE	764016651000026DZ
070131	Dalbo®-Z / Dalbo®-B / Profix / Pro-Snap Parallelometer insert	Steel	No	CE	764016651000018E2



* Manufacturer: Elos Medtech Pinol A/S, Engvej 33, DK-3330 GØRDLØSE

Table 2








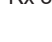










Implant system	Cat. No.	Product name	Platform	Torque (recommendation)	Basic UDI-DI
Straumann®	05002446	Dalbo® Abutment	Straumann® RN 4.8, GH1	Torque 35Ncm	764016651000046E7
	05002447	Dalbo® Abutment	Straumann® RN 4.8, GH2	Torque 35Ncm	764016651000046E7
	05002448	Dalbo® Abutment	Straumann® RN 4.8, GH3	Torque 35Ncm	764016651000046E7
	05002456	Dalbo® Abutment	Straumann® RC 4.1/4.8, GH3	Torque 35Ncm	764016651000046E7
	05002457	Dalbo® Abutment	Straumann® RC 4.1/4.8, GH4	Torque 35Ncm	764016651000046E7
	05002458	Dalbo® Abutment	Straumann® RC 4.1/4.8, GH5	Torque 35Ncm	764016651000046E7
	05002476	Dalbo® Abutment	Straumann® NC 3.3, GH3	Torque 35Ncm	764016651000046E7
	05002477	Dalbo® Abutment	Straumann® NC 3.3, GH4	Torque 35Ncm	764016651000046E7
	05002478	Dalbo® Abutment	Straumann® NC 3.3, GH5	Torque 35Ncm	764016651000046E7
Astra Tech	05002706	Dalbo® Abutment	Astra Tech OsseoSpeed® EV 3.6, GH3	Torque 25Ncm	764016651000046E7
	05002707	Dalbo® Abutment	Astra Tech OsseoSpeed® EV 3.6, GH4	Torque 25Ncm	764016651000046E7
	05002708	Dalbo® Abutment	Astra Tech OsseoSpeed® EV 3.6, GH5	Torque 25Ncm	764016651000046E7
	05002716	Dalbo® Abutment	Astra Tech OsseoSpeed® EV 4.2, GH3	Torque 25Ncm	764016651000046E7
	05002717	Dalbo® Abutment	Astra Tech OsseoSpeed® EV 4.2, GH4	Torque 25Ncm	764016651000046E7
	05002718	Dalbo® Abutment	Astra Tech OsseoSpeed® EV 4.2, GH5	Torque 25Ncm	764016651000046E7
	05002726	Dalbo® Abutment	Astra Tech OsseoSpeed® EV 4.8, GH3	Torque 25Ncm	764016651000046E7
	05002727	Dalbo® Abutment	Astra Tech OsseoSpeed® EV 4.8, GH4	Torque 25Ncm	764016651000046E7
	05002728	Dalbo® Abutment	Astra Tech OsseoSpeed® EV 4.8, GH5	Torque 25Ncm	764016651000046E7
	05002736	Dalbo® Abutment	Astra Tech OsseoSpeed® 3.5/4.0, GH3	Torque 25Ncm	764016651000046E7
	05002737	Dalbo® Abutment	Astra Tech OsseoSpeed® 3.5/4.0, GH4	Torque 25Ncm	764016651000046E7
	05002738	Dalbo® Abutment	Astra Tech OsseoSpeed® 3.5/4.0, GH5	Torque 25Ncm	764016651000046E7
	05002746	Dalbo® Abutment	Astra Tech OsseoSpeed® 4.5/5.0, GH3	Torque 25Ncm	764016651000046E7
	05002747	Dalbo® Abutment	Astra Tech OsseoSpeed® 4.5/5.0, GH4	Torque 25Ncm	764016651000046E7
	05002748	Dalbo® Abutment	Astra Tech OsseoSpeed® 4.5/5.0, GH5	Torque 25Ncm	764016651000046E7
Dentsply	05002756	Dalbo® Abutment	DENTSPLY Ankylos® C 3.5,4.5,5.5,7.0, GH3	Torque 25Ncm	764016651000046E7
	05002757	Dalbo® Abutment	DENTSPLY Ankylos® C 3.5,4.5,5.5,7.0, GH4	Torque 25Ncm	764016651000046E7
	05002758	Dalbo® Abutment	DENTSPLY Ankylos® C 3.5,4.5,5.5,7.0, GH5	Torque 25Ncm	764016651000046E7
Nobel Biocare	05002486	Dalbo® Abutment	Nobel Biocare Replace Select® RP 4.3, GH1	Torque 35Ncm	764016651000046E7
	05002487	Dalbo® Abutment	Nobel Biocare Replace Select® RP 4.3, GH2	Torque 35Ncm	764016651000046E7
	05002488	Dalbo® Abutment	Nobel Biocare Replace Select® RP 4.3, GH3	Torque 35Ncm	764016651000046E7
	05002496	Dalbo® Abutment	Nobel Biocare Replace Select® NP 3.5, GH1	Torque 35Ncm	764016651000046E7
	05002497	Dalbo® Abutment	Nobel Biocare Replace Select® NP 3.5, GH2	Torque 35Ncm	764016651000046E7
	05002498	Dalbo® Abutment	Nobel Biocare Replace Select® NP 3.5, GH3	Torque 35Ncm	764016651000046E7
	05002506	Dalbo® Abutment	Nobel Biocare Active® RP 4.3/5.0, GH3	Torque 35Ncm	764016651000046E7
	05002507	Dalbo® Abutment	Nobel Biocare Active® RP 4.3/5.0, GH4	Torque 35Ncm	764016651000046E7
	05002508	Dalbo® Abutment	Nobel Biocare Active® RP 4.3/5.0, GH5	Torque 35Ncm	764016651000046E7
	05002516	Dalbo® Abutment	Nobel Biocare Active® NP 3.5, GH3	Torque 35Ncm	764016651000046E7
	05002517	Dalbo® Abutment	Nobel Biocare Active® NP 3.5, GH4	Torque 35Ncm	764016651000046E7
	05002518	Dalbo® Abutment	Nobel Biocare Active® NP 3.5, GH5	Torque 35Ncm	764016651000046E7
	05002526	Dalbo® Abutment	Nobel Biocare Brånemark® RP 4.0, GH3	Torque 35Ncm	764016651000046E7
	05002527	Dalbo® Abutment	Nobel Biocare Brånemark® RP 4.0, GH4	Torque 35Ncm	764016651000046E7
	05002528	Dalbo® Abutment	Nobel Biocare Brånemark® RP 4.0, GH5	Torque 35Ncm	764016651000046E7
Osstem®	05002796	Dalbo® Abutment	Osstem® US Regular 4.1, GH3	Torque 30Ncm	764016651000046E7
	05002797	Dalbo® Abutment	Osstem® US Regular 4.1, GH4	Torque 30Ncm	764016651000046E7
	05002798	Dalbo® Abutment	Osstem® US Regular 4.1, GH5	Torque 30Ncm	764016651000046E7
	05002906	Dalbo® Abutment	Osstem® TS Regular 4.0/4.5/5.0/6.0/7.0, GH3	Torque 30Ncm	764016651000046E7
	05002907	Dalbo® Abutment	Osstem® TS Regular 4.0/4.5/5.0/6.0/7.0, GH4	Torque 30Ncm	764016651000046E7
	05002908	Dalbo® Abutment	Osstem® TS Regular 4.0/4.5/5.0/6.0/7.0, GH5	Torque 30Ncm	764016651000046E7
	05002916	Dalbo® Abutment	Osstem® TS Mini 3.5, GH3	Torque 30Ncm	764016651000046E7
	05002917	Dalbo® Abutment	Osstem® TS Mini 3.5, GH4	Torque 30Ncm	764016651000046E7
	05002918	Dalbo® Abutment	Osstem® TS Mini 3.5, GH5	Torque 30Ncm	764016651000046E7
	05002926	Dalbo® Abutment	Osstem® SS Regular 4.8, GH1	Torque 30Ncm	764016651000046E7
	05002927	Dalbo® Abutment	Osstem® SS Regular 4.8, GH2	Torque 30Ncm	764016651000046E7
	05002928	Dalbo® Abutment	Osstem® SS Regular 4.8, GH3	Torque 30Ncm	764016651000046E7
	Camlog®	05002766	Dalbo® Abutment	Camlog® 3.8, GH1	Torque 30Ncm
05002767		Dalbo® Abutment	Camlog® 3.8, GH2	Torque 30Ncm	764016651000046E7
05002768		Dalbo® Abutment	Camlog® 3.8, GH3	Torque 30Ncm	764016651000046E7
05002776		Dalbo® Abutment	Camlog® 4.3, GH1	Torque 30Ncm	764016651000046E7
05002777		Dalbo® Abutment	Camlog® 4.3, GH2	Torque 30Ncm	764016651000046E7
05002778		Dalbo® Abutment	Camlog® 4.3, GH3	Torque 30Ncm	764016651000046E7
05002786		Dalbo® Abutment	Conelog® 3.8/4.3, GH3	Torque 30Ncm	764016651000046E7

Implant system	Cat. No.	Product name	Platform	Torque (recom- mendation)	Basic UDI-DI
	05002787	Dalbo® Abutment	Conelog® 3.8/4.3, GH4	Torque 30Ncm	764016651000046E7
	05002788	Dalbo® Abutment	Conelog® 3.8/4.3, GH5	Torque 30Ncm	764016651000046E7
Zimmer	05002946	Dalbo® Abutment	Zimmer Tapered Screw-Vent® 3.5, GH3	Torque 30Ncm	764016651000046E7
	05002947	Dalbo® Abutment	Zimmer Tapered Screw-Vent® 3.5, GH4	Torque 30Ncm	764016651000046E7
	05002948	Dalbo® Abutment	Zimmer Tapered Screw-Vent® 3.5, GH5	Torque 30Ncm	764016651000046E7
	05002936	Dalbo® Abutment	Zimmer Tapered Screw-Vent® 4.5, GH3	Torque 30Ncm	764016651000046E7
	05002937	Dalbo® Abutment	Zimmer Tapered Screw-Vent® 4.5, GH4	Torque 30Ncm	764016651000046E7
	05002938	Dalbo® Abutment	Zimmer Tapered Screw-Vent® 4.5, GH5	Torque 30Ncm	764016651000046E7
MIS® (wide platform)	05002946	Dalbo® Abutment	MiS® Seven 3.5, GH3	Torque 30Ncm	764016651000046E7
	05002947	Dalbo® Abutment	MiS® Seven 3.5, GH4	Torque 30Ncm	764016651000046E7
	05002948	Dalbo® Abutment	MiS® Seven 3.5, GH5	Torque 30Ncm	764016651000046E7
	05002936	Dalbo® Abutment	MiS® Seven 4.5, GH3	Torque 30Ncm	764016651000046E7
	05002937	Dalbo® Abutment	MiS® Seven 4.5, GH4	Torque 30Ncm	764016651000046E7
	05002938	Dalbo® Abutment	MiS® Seven 4.5, GH5	Torque 30Ncm	764016651000046E7
BioHorizons®	05002946	Dalbo® Abutment	BioHorizons® Internal 3.5, GH3	Torque 30Ncm	764016651000046E7
	05002947	Dalbo® Abutment	BioHorizons® Internal 3.5, GH4	Torque 30Ncm	764016651000046E7
	05002948	Dalbo® Abutment	BioHorizons® Internal 3.5, GH5	Torque 30Ncm	764016651000046E7
	05002936	Dalbo® Abutment	BioHorizons® Internal 4.5, GH3	Torque 30Ncm	764016651000046E7
	05002937	Dalbo® Abutment	BioHorizons® Internal 4.5, GH4	Torque 30Ncm	764016651000046E7
	05002938	Dalbo® Abutment	BioHorizons® Internal 4.5, GH5	Torque 30Ncm	764016651000046E7
Sweden+Martina	05002956	Dalbo® Abutment	Sweden+Martina Premium Kohno 3.3, GH3	Torque 30Ncm	764016651000046E7
	05002957	Dalbo® Abutment	Sweden+Martina Premium Kohno 3.3, GH4	Torque 30Ncm	764016651000046E7
	05002958	Dalbo® Abutment	Sweden+Martina Premium Kohno 3.3, GH5	Torque 30Ncm	764016651000046E7
	05002966	Dalbo® Abutment	Sweden+Martina Premium Kohno 3.8, GH3	Torque 30Ncm	764016651000046E7
	05002967	Dalbo® Abutment	Sweden+Martina Premium Kohno 3.8, GH4	Torque 30Ncm	764016651000046E7
	05002968	Dalbo® Abutment	Sweden+Martina Premium Kohno 3.8, GH5	Torque 30Ncm	764016651000046E7
	05002976	Dalbo® Abutment	Sweden+Martina Premium Kohno 4.25, GH3	Torque 30Ncm	764016651000046E7
	05002977	Dalbo® Abutment	Sweden+Martina Premium Kohno 4.25, GH4	Torque 30Ncm	764016651000046E7
	05002978	Dalbo® Abutment	Sweden+Martina Premium Kohno 4.25, GH5	Torque 30Ncm	764016651000046E7
	05002986	Dalbo® Abutment	Sweden+Martina Premium Kohno 5.0/6.0, GH3	Torque 30Ncm	764016651000046E7
	05002987	Dalbo® Abutment	Sweden+Martina Premium Kohno 5.0/6.0, GH4	Torque 30Ncm	764016651000046E7
	05002988	Dalbo® Abutment	Sweden+Martina Premium Kohno 5.0/6.0, GH5	Torque 30Ncm	764016651000046E7

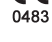30 Symbols

-  Important information for the specialist
-  Warning symbol for increased caution

Labelling on packaging/symbols

-  Date of manufacture
-  Manufacturer
-  Catalogue number
-  Batch code
-  Quantity
-  Observe the instructions for use, which are available in electronic form at the address specified.
www.cmsa.ch/docs
-  Rx only
Attention: According to US federal law, this product may only be sold by or on behalf of a physician.
-   Cendres+Métaux products with CE labelling meet the requirements of the relevant European requirements.
-  Do not re-use
-  Non-sterile
-  Keep away from sunlight
-  Attention, observe accompanying documents
-   Unique Device Identification – UDI
-  European Authorised Representative
-  Importer in EU
-  Medical device