








JD ICON ultra, s

SMALLER AND STRONGER

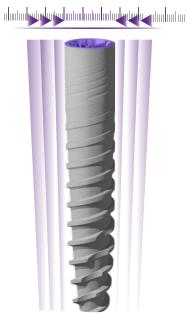

Ø 2,75mm

JDIcon Ultra S is a two-piece implant with 2.75mm diameter and internal conical connection.

It is one of the smallest implant with internal connection ever manufactured.

Despite its small diameter JDIcon Ultra.S provides a high strength due to its design and its tight conical connection.

PRODUCT SPECIFICATIONS

The JDIcon Ultra. S dental implants are available in the lengths shown in the following table:

IMPLANT DIAMETER	TIP DIAMETER	ABUTMENT INFERFACE	PLATFORM DIAMETER		L	ENGT	Н	
Ø 2,75	1,95	2,3	2,75	8	10	11,5	13	15

Note: all measurements in mm






DRILLING SEQUENCE

It is recommended to adhere to the indications of the following drilling sequence to ensure optimal primary stability of the implant.

IMPLANT	SOFT BONE	MEDIUM BONE	DENSE BONE
DIAMETER	TYPE IV	TYPE II-III	TYPE I
Ø 2,75	1,5 2,0	2,0 2,4	

*Until the first laser mark **Note:** all measurements in mm

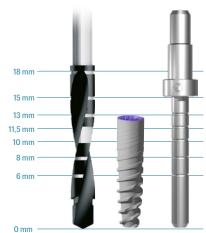

Important:

PRODUCT CATALOGUE

Implants:

Ø 2,75

 IC27080
 JDIcon Ultra S Ø 2.75 L 8

 IC27100
 JDIcon Ultra S Ø 2.75 L 10

 IC27115
 JDIcon Ultra S Ø 2.75 L 11.5

 IC27130
 JDIcon Ultra S Ø 2.75 L 13

 IC27150
 JDIcon Ultra S Ø 2.75 L 15



Cover Screw:

ISCS Cover Screw JDIcon Ultra S

Healing Abutment:

ISHA323 Healing Abutment Ø 3.2 H 3.0 JDIcon Ultra S ISHA325 Healing Abutment Ø 3.2 H 5.0 JDIcon Ultra S ISHA327 Healing Abutment Ø 3.2 H 7.0 JDIcon Ultra S ISHA403 Healing Abutment Ø 4.0 H 3.0 JDIcon Ultra S ISHA405 Healing Abutment Ø 4.0 H 5.0 JDIcon Ultra S ISHA407 Healing Abutment Ø 4.0 H 7.0 JDIcon Ultra S




Impression Coping:

OPEN TRAY

ISICOT32C Impression Coping Open Tray Ø 3.2 JDIcon Ultra S
ISICOT40C Impression Coping Open Tray Ø 4.0 JDIcon Ultra S

CLOSED TRAY

ISICCT32C Impression Coping Closed Tray Ø 3.2 JDIcon Ultra S ISICCT40C Impression Coping Closed Tray Ø 4.0 JDIcon Ultra S

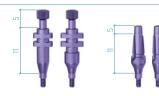

Prosthetic Screw:

ISS Prosthetic Screw JDIcon Ultra S

ISSA Prosthetic Screw Angulated JDIcon Ultra S





Temporary Abutment:

ISTAEC Temporary Abutment Engaging JDIcon Ultra S

Torque recommended 20 Ncm



GP Abutment:

ISGP32C GP Abutment Ø 3.2 JDIcon Ultra S ISGP40C GP Abutment Ø 4.0 JDIcon Ultra S IS15GPA40C GP Abutment 15° Ø 4.0 JDIcon Ultra S

Torque recommended 20 Ncm



Cobalt-Chrome Abutment:

ISCCEC Cobalt-Chrome Abutment Engaging JDIcon Ultra S

EVCACCNEC Cobalt-Chrome Conical Abutment Non Engaging JDEvolution

Torque recommended 20 Ncm



Wax-Up Abutment:

ISWAEC Wax-Up Abutment Engaging JDIcon Ultra S

Torque recommended 20 Ncm



Straight Abutment:

ISSA4020C Straight Abutment Ø 4.0 H 2.0 JDlcon Ultra S ISSA4040C Straight Abutment Ø 4.0 H 4.0 JDlcon Ultra S

Torque recommended 20 Ncm



Conical Abutment:

ISCA15C Conical Abutment H 1.5 JDIcon Ultra S ISCA1725C Conical Abutment Angulated 17° H 2.5 JDIcon

ISCA1725C Conical Abutment Angulated 17° H 2.5 JDIcon Ultra S ISCASA Conical Abutment Angulated Screw JDIcon Ultra S





TiBase, JDScanBody and Implant Replica CAD CAM:

ISSBCECScanBody Engaging JDIcon Ultra SEVCASBCECScanBody Conical Abutment JDEvolutionEVCASBCECScanBody Engaging Conical Abutment

EVCASBCESC: On Top ScanBody Conical Abutment JDEvolution Plus*

EVSCBOW Scan Body for TiBase
ISTIBC TiBase JDIcon Ultra S
ISTIBNC TiBase New JDIcon Ultra S
ISTIB15NC TiBase New H 1.5 JDIcon Ultra S
ISTIB30NC TiBase New H 3.0 JDIcon Ultra S

EVCAITEC Interface CAD CAM for Conical Abutment Engaging JDEvolution (H 7.5)

EVA139C TiBase Engaging for Conical Abutment (H 5.0)

EVCAITC Interface CAD CAM for Conical Abutment Non Engaging JDEvolution (H 7.3)

EVCATIBC TiBase for Conical Abutment JDEvolution (H 5.0)
ISANCN Implant Replica CAD CAM New JDIcon Ultra S
ISPMNC Pre-milled Abutment Blank JDIcon Ultra S

*for screw use JDEvolution plus prosthetic screwdriver





Torque recommended 20 Ncm



Torque recommended 15 Ncm



Torque recommended 20 Ncm



Ball Abutment:

ISBA15Ball Abutment H 1.5 JDIcon Ultra SISBA30Ball Abutment H 3.0 JDIcon Ultra SISBA50Ball Abutment H 5.0 JDIcon Ultra S

EVBAA Ball Abutment Replica

EVBAHC Ball Abutment Elastic Retention Cap w/ Container

EVBAN Ball Abutment Retention Cap



Torque recommended 20 Ncm



Implant Replica:

ISAN Implant Replica JDIcon Ultra S

Implant and Prosthetic Drivers:

ISID ISIDL	Implant Driver JDIcon Ultra S Implant Driver Long JDIcon Ultra S	
EVSDPF15 EVSDPF20 EVSDPF25 EVSDPF35 EVSDPF60	Prosthetic Driver for JDTorque L 15 Prosthetic Driver for JDTorque L 20 Prosthetic Driver for JDTorque L 25 Prosthetic Driver for JDTorque L 35 Prosthetic Driver for JDTorque L 60	
EVSDCAF	Conical Abutment Driver for JDTorque	
EVSDP20 EVSDP25 EVSDP30	Prosthetic Driver L 20 Prosthetic Driver L 25 Prosthetic Driver L 30	
EVSDCA	Conical Abutment Driver	<u> </u>
EVSDPF25A EVSDPF30A	Angulated Screw Driver for JDTorque L 25 Angulated Screw Driver for JDTorque L 30	
JDG ISGMC	JDGuide Implant Mounter for JDIcon Ultra S	

Direction inticators:

JDDI Direction Indicator
JDDIS Direction Indicator Short
JDDI17 Direction Indicator 17°

